

## Warnings and cautions

### Post-treatment procedure

**IMPORTANT!** A successful washing is directly verified by visualizing the tympanic membrane before the disappearance of the ear wax plug, and indirectly by detecting earwax in the remainder removed during the maneuver.

**IMPORTANT!** After removing the ear wax, dry the auricular pavilion.

**IMPORTANT!** In case of trouble with the complete removal of the ear wax plug after irrigating the recommended 400ml per session, it is advised to install earwax softeners in the ear canal and repeat the irrigation on another day.

**IMPORTANT!** Empty the water device completely and dry off the device thoroughly before storage.