

Warnings and cautions

Pre-treatment procedure

IMPORTANT! Do not use the Lantitaps in case of a perforation in the tympanic membrane and/or inflammation of the ear canal.

IMPORTANT! The device cannot be used in case of pain or difficulty in previous irrigations of the ear canal.

IMPORTANT! Before the irrigation with Lantitaps, it is advisable to check by direct inspection the presence of ear plugs in the ear canal.

IMPORTANT! Even if dispensable, the use of ear wax softeners is recommended before irrigating the ear, at least 15 minutes before the maneuver, to reduce the number of irrigations needed to clean the ear canal successfully, thus improving the effectiveness of Lantitaps.

IMPORTANT! When a professional uses Lantitaps for the first time, and before using it with a patient, it is recommended to fill with water and empty the reservoir a couple of times to check the water outlet pressure.

IMPORTANT! If the device is not used frequently, do several irrigations directly on the fingertips to test the water outlet pressure before performing the irrigation in the ear.